HQ-AT10

MANUAL (p. 2)
ALCOHOL BREATH TESTER

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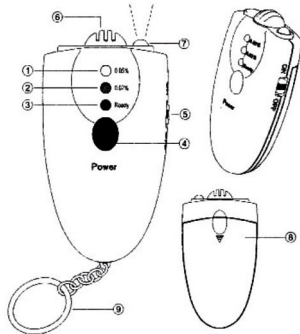
- 1)--Red Light (Danger)
- 2)--Yellow Light (Caution)
- 3)--Green Light (OK)

2. Alcohol Test

- 2) Press and hold the power button(4) until the green LED light turns on and hold it for 5 seconds. Then blow into the breath pipe (6) and check the test result (user's alcohol level) as follows:

A-- Green light on: Ok safe----Under 0.02% BAC (or 0.1mg/l BRAC)

BAC -----Blood Alcohol Content

BRAC ---Breath Alcohol Content

3. Torch Function

Set the torch switch (5) to ON position to turn on the light. Do not use the test function whilst the torch is on.

4. Notice

If this Alcohol Breath Tester is not used for a long period, it may need more time to warm up. Please press the power button and retry it for several times.

5. Disclaimer

The indication of BAC by this Alcohol Breath Tester should be used for reference only. The manufacturer and the seller do not take any legal responsibility.

DECLARATION OF CONFIRMITY

We,
Nedis B.V.
De Tweeling 28
5215MC 's-Hertogenbosch
The Netherlands
Tel.: 0031 73 599 1055
Email: info@nedis.com

Declare that product:
Name: HQ
Model: HQ-AT10
Description: Alcohol breath tester

Is in conformity with the following standards
EMC: EN 55022 : 1998
EN 55024 : 1998 (EN 61000-4-2/-3)

Following the provisions of the 89/336/EEC directive.
's-Hertogenbosch, 2008, February 26

Mrs. J. Gilad
Purchase Director

Safety precautions:

mains and other equipment if a problem should occur. Do not expose the product to water or moisture.

To reduce risk of electric shock, this product should **ONLY** be opened by an authorized technician when service is required. Disconnect the product from

Maintenance:

Clean only with a dry cloth. Do not use cleaning solvents or abrasives.

Warranty:

No guarantee or liability can be accepted for any changes and modifications of the product or damage caused due to incorrect use of this product.

General:

Designs and specifications are subject to change without notice.


All logos brands and product names are trademarks or registered trademarks of their respective holders and are hereby recognized as such.

**Attention:**

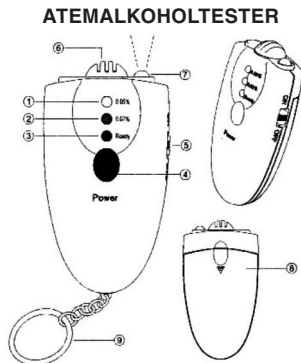This product is marked with this symbol. It means that used electrical and electronic products should not be mixed with general household waste.

There is a separate collections system for these products.

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Copyright ©
Copyright © (Autorská práva)