

## Applications:

- Cystectomy
- Total Abdominal Hysterectomy
- Myomectomy
- Tubal Ligation
- Lapcolectomy
- Gastric Bybas
- Thoratomy - paraesophageal hiatal hernia
- Appendectomy
- Splenectomy

## OR/Mobius

| PRODUCT NUMBER / DESCRIPTION: |                                       |                                  |
|-------------------------------|---------------------------------------|----------------------------------|
| 900-508                       | Mini Mobius<br>inner diameter 8 cm    | Optimal incision<br>size 2-6 cm  |
| 900-500                       | Small Mobius<br>inner diameter 11 cm  | Optimal incision<br>size 4-9 cm  |
| 900-515                       | medium mobius<br>inner diameter 15 cm | Optimal incision<br>size 8-14 cm |

**Also available in larger sizes for maximizing exposure during c-sections.**

## OB/Mobius®

| PRODUCT NUMBER / DESCRIPTION: |  |                                   |
|-------------------------------|--|-----------------------------------|
| 900-517                       | Mobius, Medium-Large<br>inner diameter 17 cm | Optimal incision<br>size 9-15 cm  |
| 900-520                       | Mobius, Large<br>inner diameter 20 cm        | Optimal incision<br>size 12-17 cm |

*Mobius is latex-free, made of 100% medical grade polyurethane. It is supplied sterile, one retractor per pouch. 5/box*

## OR/Mobius®

Maximize exposure, safety and efficiency with the OR/Mobius Self-Retaining Elastic Abdominal Retractor for Minilaparotomies



## References:

- <sup>1</sup> Minimally Invasive Cesarean: Improving an Innovative Technique. Pelosi & Pelosi, OBG Management, July 2004
- <sup>2</sup> The Use of the Mobius® Self-Retaining Retractor in Cesarean Deliveries—A Retrospective Case. Finck, MD & C Della Badia, Poster, AAGL 33rd Annual Meeting, Nov. 2004



CooperSurgical is the leading company dedicated to providing medical devices and procedure solutions that improve health care delivery to women regardless of clinical setting. Our company is fostering that position through expansion of its core businesses and introduction of advanced technology-based products which aid clinicians in the management and treatment of commonly seen conditions.

203.601.5200 | 800.243.2974 | [coopersurgical.com](http://coopersurgical.com) | 95 Corporate Dr. | Trumbull, CT 06611  
also at [www.YouTube.com/CooperSurgical](http://www.YouTube.com/CooperSurgical)

Copyright © 2011 CooperSurgical

Form #82019 Rev 07/11




## Improved minilaparotomy safety, efficiency and speed are just some of the benefits of the cost-effective OR/Mobias® retractor

Once inserted, using a simple roll down technique, the Mobius® retractor provides 360 degrees of radial retraction: maximizing working area and providing significantly greater exposure than traditional retractors.


The Mobius retractor comes in three sizes optimal for general, colorectal and gynecologic surgeries. Each size distributes retraction force evenly without excess pressure around the incision, creating unobstructed exposure to the surgical field. Throughout the procedure this device maximizes working area, minimizes trauma, reduces risk for wound infection, blood loss and allows for faster patient recovery.

OB/Mobius retractors are also available for use in c-section procedures. For more information please visit [www.coopersurgical.com/obmobius](http://www.coopersurgical.com/obmobius)

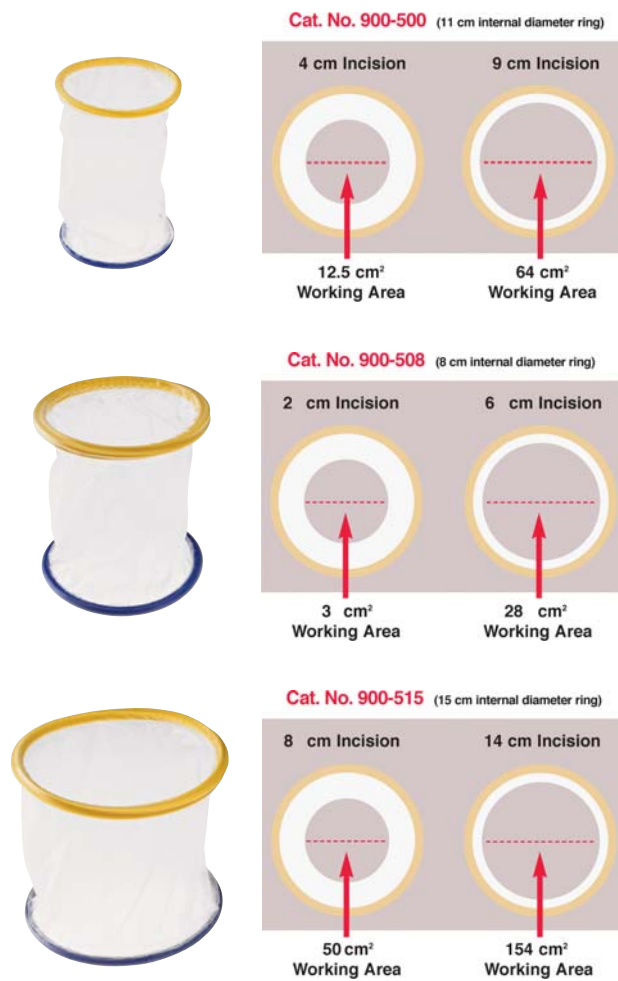

### Simple Roll-down Technique:

1. Squeeze the lower blue ring and insert into the abdomen. Gently pull the ring to ensure insertion and perform a quick digital exam to check for trapped tissue.
2. Position hands at 11 and 1 o'clock with assistant at the 12 o'clock position.
3. Roll down the ring outwards first with the left, followed by the right, slide fingers down the ring to complete a Mobius flip. Repeat process 2-3 times until desired exposure achieved.

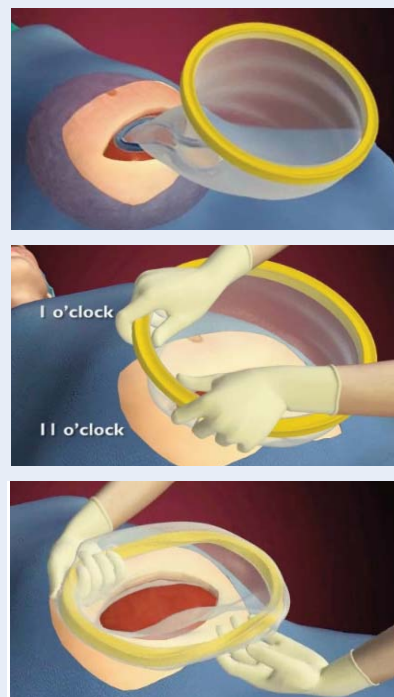

## A better retractor means better visualization.


Our patented outer ring provides better flexibility and form.

### Better Flexibility

- Outer ring lies flush against patient's skin without excessive pressure reducing pain, potentially reducing blood loss.
- Outer ring lies flush against patient's skin without interfering with surgeon's wrists.
- Flexible polyurethane sheath maintains moisture at the incision site while reducing risk of wound infection.

### Better Form

- One-piece, molded outer ring construction rolls outward providing maximum hands-free exposure even in obese patients.
- Molds gently to the patient's abdomen for secure low profile retraction that won't slip.
- Only 2-3 outward rolldowns required to achieve optimal exposure.
- Tear resistant, non-stretch sleeve provides superior tissue retraction while allowing for movement through the wound.
- Bottom ring maintains shape, preserving exposure throughout procedure.

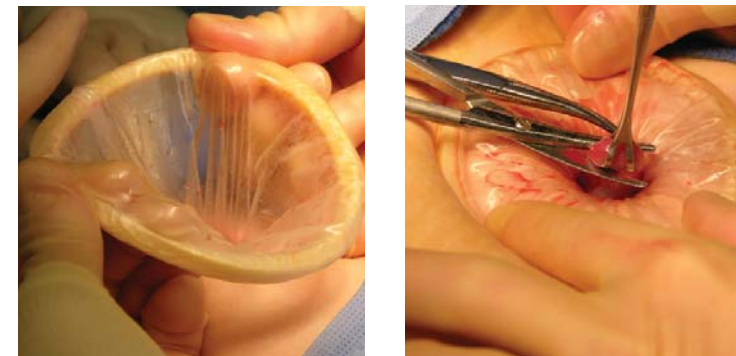

Easy to place.  
Easy to use.  
Easy to remove.

### Benefits of the Mobius:

- Maximizes exposure through radial retraction for enhanced visibility
- Presents unobstructed surgical field unencumbered by traditional stainless steel retractors
- Alleviates need for additional surgical staff
- Protects wound edges minimizing contamination<sup>1,2</sup>
- Gentle retraction through dispersed tension promotes patient postoperative comfort
- Ideal for obese patients (including BMI > 30 lbs.)
- Easy to place and remove with simple technique