# boso medicus system

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DE 2

**User  
Instructions**

GB 46

**Mode  
d'emploi**

FR 90

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IT 134

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ES 178

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# Table of contents

---

Product features.....	48
Contents of package .....	49
Quick guide .....	50
Introduction.....	51
Purpose.....	52
Blood pressure values.....	53
Systolic and diastolic blood pressure value .....	53
WHO blood pressure cut-off values .....	53
Starting up.....	55
Inserting batteries .....	55
Battery status.....	56
General instructions for self-measurement.....	57
General Notices for Bluetooth Connection.....	59
Preparing to measure your blood pressure.....	61
Note on cuff sizes .....	61
Attaching the cuff.....	62

# Table of contents

---

Measuring your blood pressure.....	65
Interrupting a measurement.....	66
Measurement display.....	67
Set up app and create connection.....	69
Measuring and transferring blood pressure.....	73
Evaluating and sending measurements.....	75
Deleting measurement values.....	77
Mains operation.....	78
Error messages.....	79
Cleaning and Disinfection.....	82
Warranty Conditions.....	83
Customer service / Disposal.....	84
Technical data.....	85
Calibration checks - Testing instructions (for trained specialist staff only).....	88

## Product features

-  Battery status display see page 56
-  Measurement in progress see page 66
-  Irregular heartbeat during measurement see page 67, 80
-   **WARNING:** To avoid the danger of suffocation, keep this bag away from babies and children. Do not use in cribs, cots, beds, carriage, prams or playpens. This bag is not a toy. see bag
-  Bluetooth communication active

# Contents of package

1 **boso medicus system**  
blood pressure monitor

4 LR 6 (AA) batteries

1 CA 04 universal cuff

1 User  
instructions

1 Bag

## Quick guide

- ➔ Insert batteries, ensure correct positioning (P. 55)
- ➔ Connect cuff to monitor (P. 61)
- ➔ Attach cuff (P. 62)
- ➔ Start (and stop) a measurement by pressing the start button (P. 65)

GB

Measured systolic, diastolic and pulse values are displayed after the measurement is completed (P. 67)

This quick guide does not replace the detailed information on the use and safety of the blood pressure monitor featured in the user instructions.

Please be sure therefore to read the user instructions!

# Introduction

Dear Customer,

We are delighted that you have decided to purchase a bosso blood pressure monitor. The bosso brand is synonymous with optimal quality and precision and is also number 1 among professionals: 96% of all German doctors currently use bosso blood pressure monitors. The experience gained from decades of use by professionals has also been incorporated into all the patient monitors for home measurement.

This instrument has also passed our strict quality control procedures and is your reliable partner for monitoring your blood pressure values.

Please read these user instructions before using the instrument for the first time, as correct blood pressure readings

can only be obtained if the instrument is operated correctly.

In these user instructions, „➡“ means an action by the user.

To get help concerning usage or maintenance please contact your dealer or the manufacturer (contact details can be found on the back cover of this manual).

An unexpected operating condition or an incidence which has worsened the health condition or could have worsened the health condition should be reported to the manufacturer immediately.

If the instrument is to be sold, ensure that these user instructions are enclosed.

## Introduction

---

If the monitor is used in medical practice (in accordance with the legislation governing the operation of medical devices), it should be calibrated at regular intervals (see page 88).

Wireless communication devices, such as home networking devices, mobile phones, cordless phones and their base stations, walkie-talkies can affect this blood pressure monitor. Therefore, a minimum distance of 11 feet should be kept from such devices. Do not use a cell phone during the blood pressure measurement.

This device conforms to the European medical products guidelines 93/42/EWG. This is confirmed by the CE marking CE

0124. (The ID number 0124 stands for the notified body DEKRA)

The Bluetooth radio transmission of the device conforms to the telecommunications guidelines 1999/5/EG.

### Purpose

Non-invasive recording of systolic and diastolic blood pressure and pulse frequency in humans.

The instrument can be used with any arm circumference as shown on the corresponding cuff.

## Blood pressure values

To determine the blood pressure, **two values** need to be measured:

- **Systolic (upper) blood pressure:**

This is produced when the heart muscle contracts and blood is forced into the blood vessels.

- **Diastolic (lower) blood pressure:**

This is present when the heart muscle expands and fills with blood again.

Blood pressure measurement values are expressed in mmHg (mm of mercury).

The World Health Organization (WHO) has defined the following cut-off values for assessing blood pressure:

	systolic	diastolic
High blood pressure (hypertension)	over 140 mmHg	over 90 mmHg
Borderline normal blood pressure	130 to 139 mmHg	85 to 89 mmHg
Normal blood pressure	120 to 129 mmHg	80 to 84 mmHg
Optimum blood pressure	up to 119 mmHg	up to 79 mmHg

## Blood pressure values

---

Whether medical treatment is necessary depends not only on the blood pressure, but also on the patient's risk profile. Please contact your GP if one of the values (systole, diastole) consistently exceeds the cut-off point for high blood pressure.

GB

Self-measurement of blood pressure often produces somewhat lower values at home

than at the doctor's. The German Hypertension League has therefore defined lower cut-off values for home measurements:

Home measurement:	135/85 mmHg
Measurement in the doctor's surgery:	140/90 mmHg

## Starting up

The device must be installed and put into operation in accordance with the instructions in this manual.

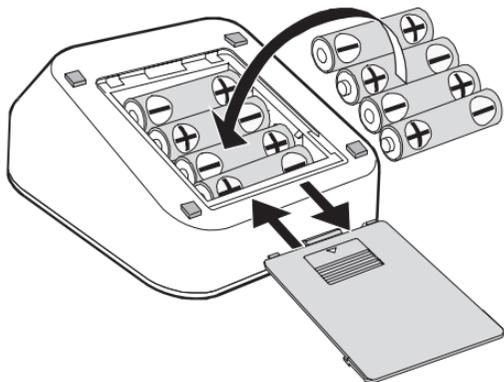
Illustration 1

➔ Use only high quality leak-proof batteries of the correct specifications (see "Technical data" P. 85).

**⚠** **Never** mix old and new batteries or different makes.

**⚠** If the batteries are not inserted correctly, the monitor will not work and this may cause the batteries to heat up and leak, damaging the monitor.

### ➔ Inserting the batteries

The battery compartment is located on the underside of the monitor. Insert batteries in the battery compartment as shown in illustration 1.

➔ Remove the batteries if the monitor is not to be used for any length of time.

## Starting up

The monitor has a battery status display (see P. 48)

Battery fully charged.

Battery partially charged.

Replace battery soon.

Flashing! No further measurements possible.  
Replace battery.

### Care for the environment

Do not dispose of used batteries and charger units in domestic waste. You can take these to collection sites for used batteries or special waste. Contact your local authority for information.

## General instructions for self-measurement

1. Fluctuations in blood pressure are quite normal. Even when measurements are repeated, there can be marked differences. Single or irregular measurements do not provide reliable information about the actual blood pressure. A reliable evaluation is only possible if regular measurements are taken under comparable conditions and the measurement values entered in the blood pressure record card.
2.  Self-testing does not mean self-treatment. Do not on **any** account change the dose of medicine prescribed by your doctor of your own accord.
3. Irregular heartbeats can affect the accuracy of the monitor or result in incorrect measurements (see page 80).
4. Incorrect measurements can also occur in patients with a weak pulse (possibly in patients wearing a pacemaker, for example). The blood pressure monitor itself has no effect on the pacemaker.
5. If you are pregnant, consult your doctor before using the monitor.
6. Always take the measurement in a calm and relaxed state.  
You are recommended to check your blood pressure twice a day – in the morning on getting up and in the evening once you have relaxed after work.
7. Always measure your blood pressure from the arm with the higher blood pressure values (where not required otherwise (see page 63)).

## General instructions for self-measurement

---

To do this, measure the blood pressure first in both arms and thereafter always from the arm with the higher blood pressure.

## General Notices for Bluetooth Connection

-  The blood pressure monitor may not be used in locations where wireless communication is prohibited, such as in aircraft or hospitals. This blood pressure monitor can have unfavorable effects on electronic devices or electrical medical devices.
- The maximum communication distance between the blood pressure monitor and the receiver equals approx. 10 meters. This distance can be reduced by the environment; therefore, verify whether the distance is short enough for transmission of data after the measurement.
- Ensure that the space between the blood pressure monitor and the mobile device is unobstructed. The communication distance can be shortened by the construction of the building or other obstacles. Communication through reinforced concrete walls in particular may not be possible under certain circumstances.
- During pairing, Bluetooth must be deactivated on all other devices within range.
- If the improbable case should occur that this blood pressure monitor disrupts other wireless communication devices, use the blood pressure monitor at a different location.

## General Notices for Bluetooth Connection

---

- Do not use the Bluetooth connection in the vicinity of an IEEE802.11g/b/n-WLAN device or other wireless communications systems (particularly in the range of 2.4 GHz) or near devices which emit electromagnetic waves (such as microwave appliances). This can cause the connection to be frequently interrupted, communication speed to be significantly reduced and errors to occur.

## Preparing to measure your blood pressure

### Note on cuff sizes:

**!** The monitor may only be used with the following cuff types. These should be chosen according to the upper arm circumference printed on them.

Type	Arm circumference	Order number
CA01	22 – 32 cm	143-4-764
CA02	32 – 48 cm	143-4-757
CA04	22 – 42 cm	143-4-765*

\* Standard cuff supplied as standard.

Attach the cuff to the monitor by pressing the plug (click) into the socket on the left side of the unit. (See illustration 2).

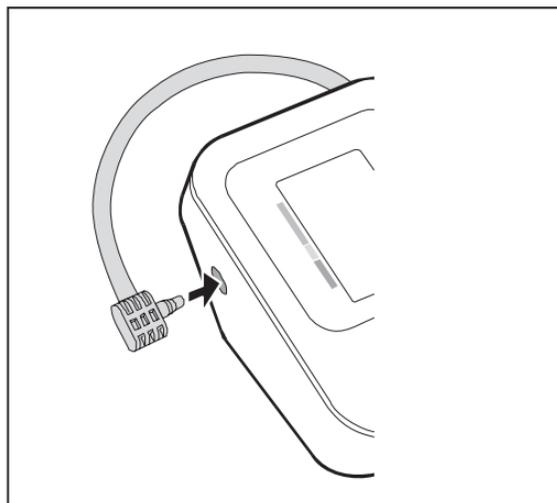
Illustration 2

## Preparing to measure your blood pressure

Observe the following instructions when measuring your blood pressure:

Avoid smoking or drinking coffee for one hour before measuring your blood pressure.

GB

Sit comfortably for the blood pressure measurement. Support your back and arms. Do not cross your legs. Place your feet flat on the floor.

Relax for 5 minutes before taking a measurement.

Do not move during the measurement.

### Attaching the cuff

The measurement should be taken from the bare upper arm.

In the case of tightly fitting outer clothing, ensure that the blood supply to the arm is not cut off when the sleeve is rolled up (if necessary remove the article of clothing before taking the measurement).

➔ Open out the cuff into the shape of a ring and slide it over the upper arm until the bottom edge of the cuff is about 2-3 cm above the elbow. The cuff must be positioned so that the marking is over the artery (see illustration 3).

## Preparing to measure your blood pressure

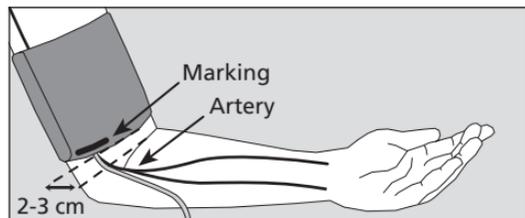
Illustration 3

⚠ Do not attach the cuff over wounds as this can cause further injuries.

⚠ Ensure that the cuff is not attached to an arm where arteries or veins are undergoing or have undergone medical treatment (e.g. shunt).

⚠ For women who have had a breast amputated, do not attach the cuff to the arm on the amputated side of the body.

⚠ During the measurement, medical apparatus that are being used on the same arm at the same time, may malfunction.

⚠ The cuff must not be too tight. There should be enough space to fit roughly two fingers between arm and cuff.

⚠ The metal ring should **never** lie over the artery (see illustration 3) as this may produce a false reading.

➔ Grasp the end of the cuff that feeds through the metal ring and wrap it around the outside of your arm. Attach the Velcro fastener by applying gentle pressure.

## Preparing to measure your blood pressure

- ➔ Place your arm with the cuff attached on the table, holding it relaxed and slightly bent so that the cuff is level with your heart.

**⚠** Check that the air tube is not kinked during the measurement. This could cause congestion of the blood flow and subsequent injury.

GB

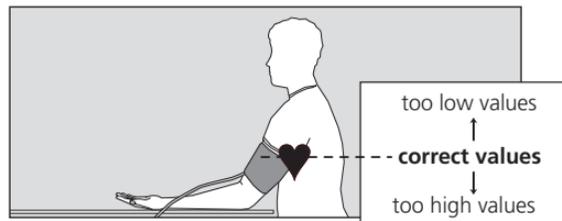
Illustration 4

**⚠** The blood pressure measurement should not prevent the flow of blood for an unnecessary length of time (> 2 minutes). If the monitor fails to function correctly, remove the cuff from the arm.

**⚠** Excessive frequent measurements can cause injury by restricting blood flow.

Do not talk during the measurement.

## Measuring your blood pressure

- ➔ Start the measurement with the start button

- ⚠ Now hold the arm absolutely still and do not talk.

All the items on the LCD display appear briefly to indicate that the monitor is now ready for use (see illustration 5).

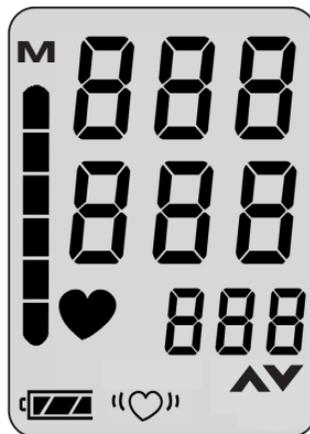
Illustration 5

## Measuring your blood pressure

The pump starts to inflate the cuff.

The monitor has an intelligent automatic inflation system for gentle inflation to the correct cuff pressure.

The rising cuff pressure is displayed as a digital numerical value and in the form of a bar on the left-hand edge of the display.

When the **required pressure** is reached, the pump switches off and the air is released slowly from the cuff.

### **Interrupting a measurement:**

A measurement can be interrupted at any stage by pressing the start button; the cuff deflates automatically.

During the measurement phase the descending cuff pressure and the flashing  symbol are displayed (see illustration 6).

Illustration 6

## Measurement display

After the end of the measurement, the in-built valve opens automatically and the cuff deflates rapidly.

The measured blood pressure values (systole, diastole, pulse) are displayed (see illustration 7).

Illustration 7

If the symbol "♡" appears after the measurement, it is recommended that the measurement is repeated, holding the arm completely still. If the "♡" symbol appears again even with the arm held still, this may be an indication of an irregular heartbeat. Please discuss this with your doctor at your next visit.

Blood pressure is a dynamic parameter and can be affected by the patient's position, e.g. sitting, standing, lying, moving, before or during the measurement, physical condition (stress, disease, etc.).

➔ If the measurement result is obviously wrong, repeat the measurement.

## Measurement display

---

The unit switches itself off automatically after about 1 min. To repeat a measurement, press the start button again **briefly**.

Allow at least two minutes between measurements.

A dark grey square containing a white circle with the letters 'GB' inside.

➔ If no further measurement is required, remove the cuff from the arm.

## Set up app and create connection

Requires interaction with boso-App

Requires interaction with boso medicus system

To enable data exchange between the blood pressure monitor and your mobile device, follow these steps:

– Ensure that Bluetooth is activated on your mobile device. (Bluetooth 4.0 is required)

– Download the boso app from the Google/Apple Store.

When installing the app, the system will ask for permission to access the current location. You must answer "YES" in order to avoid BT connection errors.

# Set up app and create connection

- Open the boso app on your mobile device.
- Set up a new user:
  1. Select "User" from the main menu.
  2. Select "+New" from the bottom menu bar.
  3. Enter your personal data. Compulsory fields are indicated by "\*" .
  4. Click on the "Save" button in the bottom menu bar to save the data you have entered.
- Tick the user for whom you wish to save the measurement values.

GB

## Set up app and create connection

- Use the “Back” button in the top menu bar to return to the main menu.
- Open “Devices” in the main menu.

- Press the START key on the blood pressure monitor and hold it down until “Pr” appears on the display. The communication indicator is shown on the display for approximately one minute.

- In the “Devices” menu, select “Add new device” in the bottom menu bar.

If you are using iOS, you must also press “Pair” to confirm the pairing request.

- Date and time are automatically synchronised by the mobile device.

- After pairing successfully, “End” appears in the display on the monitor and the monitor is displayed in the device list in the app.

Setup is now complete. Now leave the menu via the “Back” button in the top menu bar.

## Set up app and create connection

---

The blood pressure monitor must be reconnected to the mobile device **after changing the batteries**, as the time information is deleted when the batteries are changed.

If this is not done, the current system time of the mobile device is stored as the time stamp for the readings during data transfer.

Please proceed as follows:

- Delete the blood pressure monitor from the devices list in the app
- Delete the blood pressure monitor from the Bluetooth settings of the mobile device
- Pair again as described.

## Measuring and transferring blood pressure

Now put on the cuff and carry out a blood pressure measurement as described in the user instructions.

As long as Bluetooth is active and the app is running, the measured values are automatically transferred after the blood pressure measurement has been completed.

After successful transfer, the measurement value is deleted from the measurement value memory of the blood pressure monitor.

Repeat the pairing process if the measurement data have not been transferred. If the app or Bluetooth are inactive, up to 30 measurement

values are automatically saved in the blood pressure monitor (If the measurement value memory of 30 measurements is full, the oldest measurement is deleted and the current measurement is saved. Faulty measurements are not saved) and sent after the next measurement once the app has been activated.

In order to enable all saved measurement values in the medicus system measurement value memory to be transferred to the app, measurement value transfer of all measurement values must be activated in the "Settings" menu (default setting). If "last" is selected here,

# Measuring and transferring blood pressure

only the last measured measurement value is transferred and all saved measurement values are deleted.

Times for the morning and afternoon evaluations can also be altered in this menu.

Save your settings by using the "Save" button in the bottom menu bar.

"bosogramm system" is intended for a scale.

The screenshot shows the 'Einstellungen' (Settings) screen in the 'boso' application. At the top, there is a 'Zurück' (Back) button and the 'boso' logo. The settings are organized into sections:

- Allgemeine Einstellungen**: Includes 'Beginn Vormittag Auswertung' set to 00:00 and 'Beginn Nachmittag Auswertung' set to 12:00.
- boso medicus system**: Features a 'Messwertübernahme' (Measurement Transfer) section with two options: 'letzter' (last) and 'alle' (all). The 'alle' option is currently selected and highlighted with a dark background.
- bosogramm system**: Features a 'Modus' (Mode) section with two options: 'singleuser' and 'multiuser'. The 'multiuser' option is currently selected and highlighted with a dark background.

At the bottom of the screen, there is a navigation bar with two buttons: 'Abbrechen' (Cancel) and 'Speichern' (Save).

# Evaluating and sending measurements


All measurement values that have been sent so far are displayed in the "Measurement values" menu.

Use the "Graphs" button in the bottom menu bar to see a graphic representation of the measurement values.

The buttons in the top area of the screen allow you to select the graphs available: "SYS/DIA" and "PULSE". Click on a graph to enlarge or shrink it.

"kg/BMI" is intended for scales.

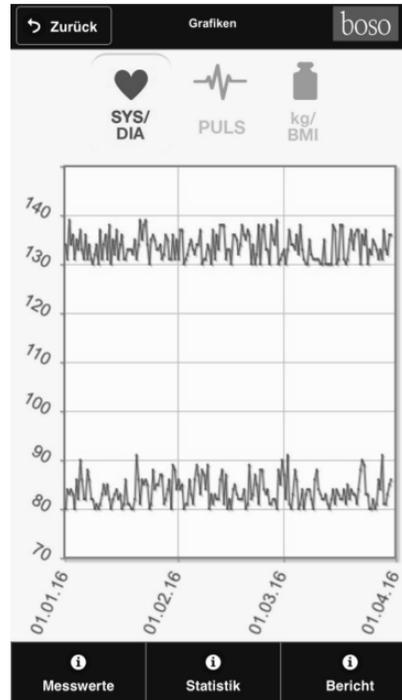# Evaluating and sending measurements

Use the "Statistics" button in the bottom menu bar to see a statistical summary of the measurement values. The time range for the summary (date from/to) can be selected.

GB

The "Report" button in the bottom menu bar takes you to the menu which you use to send measurement values. The summary can be sent by email in PDF or XML format. The XML file can be imported into the PC program "boso profilemanager XD Home." This program offers additional evaluation options and is available free of charge from the Internet (<http://www.boso.de/download/profilmanagerXDHome/>).

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An:

Kopie:

Email  
Text:

Als XML senden Als PDF senden

# Deleting measurement values

Individually selected measurement values or measurement data older than 1/3/6/12 months can be deleted via the “Delete” button in the bottom menu bar.

Note that data cannot be restored. Please archive your data in good time by creating an XML or PDF file using the “Report” button.

The screenshot shows the 'Messwerte' (Measurements) screen of the 'boso' application. At the top, there is a 'Zurück' (Back) button and the 'boso' logo. Below the header, there is a clock icon and a table of measurement data. The table has columns for 'SYS', 'DIA', 'PUL', 'Arr', 'kg', and a checkbox for deletion. The bottom menu bar contains four buttons: 'Grafiken' (Charts), 'Statistik' (Statistics), 'Bericht' (Report), and 'Löschen' (Delete).

	SYS	DIA	PUL	Arr	kg	
30.03.2016 18:00:00	-	-	-	-	78,7	<input checked="" type="checkbox"/>
30.03.2016 18:00:00	136	86	66	-	-	<input checked="" type="checkbox"/>
30.03.2016 08:00:00	-	-	-	-	78,5	<input type="checkbox"/>
30.03.2016 08:00:00	136	85	68	-	-	<input type="checkbox"/>
29.03.2016 18:00:00	-	-	-	-	78,8	<input checked="" type="checkbox"/>
29.03.2016 18:00:00	132	83	67	-	-	<input type="checkbox"/>
29.03.2016 08:00:00	-	-	-	-	78,7	<input type="checkbox"/>
29.03.2016 08:00:00	133	81	65	-	-	<input type="checkbox"/>
28.03.2016 18:00:00	-	-	-	-	78,9	<input type="checkbox"/>

## Mains operation

---

The mains connection socket is located at the rear of the monitor.

Where required, use only the bosco power supply unit (**order number 410-7-150**). This power supply unit produces a rectified output of the correct polarity. Other commercially available power supply units can cause damage to the electronic components, which will invalidate the warranty. There is also a risk of fire with these other power supply units.

## Error messages

If any problems occur during the measurement, the display will show an error message instead of a blood pressure reading.

Meaning of error messages:

	Cause of <b>Err</b> message:	Troubleshooting:
	Excessive variation in cuff pressure during measurement	Hold arm still.
	No valid pulse readings	Check position of cuff (see page 61) and repeat measurement.
	Systole – diastole difference too small: Systole – diastole $\leq 10$ mmHg	Check position of cuff (see page 61) and repeat measurement.
	Incorrectly inflated	Cuff possibly too loose, attach cuff more firmly (see page 61)

## Error messages

GB

Cause of <b>Err</b> message:		Troubleshooting:
	The pulse value could not be determined correctly.	Check position of cuff and repeat measurement.
	Internal Error	Remove the batteries and briefly press START button. Put the batteries back into the monitor. If the error persists, return the instrument for repairs to the manufacturer.
		
	“♥” Display:	<b>Troubleshooting:</b> Repeat measurement recommended, holding the arm absolutely still. If the “♥” symbol appears again, even with the arm held still, this may be an indication of an irregular heartbeat. Please discuss this with your doctor at your next visit.

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## Cause of **Err** message: Troubleshooting:

Bluetooth connection error

Remove the batteries and briefly press START button. Put the batteries back into the monitor. Carry out the pairing again. If the error persists, return the instrument for repairs to the manufacturer.

When installing the app, the system will ask for permission to access the current location. You must answer "YES" in order to avoid BT connection errors.

## Cleaning and Disinfection

---

### **Cleaning Monitor:**

Use a soft, dry cloth to clean your monitor.

### **Cleaning Cuff:**

Small stains on the cuff can be removed carefully with a proprietary washing-up liquid.

### **Disinfection Cuff:**

For disinfectant wipes (at least 5 minutes exposure time) of the device and the cuff, we recommend the disinfectant antiseptic liquid (Schülke & Mayr). To disinfect the cuff, we recommend spray disinfection.

In particular, if the device is used on multiple users it is to pay attention to regular cleaning and disinfections of the cuff.

## Warranty Conditions

---

We give 3 years warranty from the date of purchase. The purchase date has to be proven by the invoice. Within the warranty period defects are eliminated free of charge. After repairs the warranty period is not extended on the whole unit, but only to the replaced components.

Excluded from the warranty are parts subject to normal wear and tear (e.g. cuff), transport damages and any damage caused by improper handling (e.g. non-compliance with the instructions for use). Damages due to disassembly by unauthorized persons are also excluded from warranty.

No claims for damages against us are substantiated by the warranty.

In the case of justified warranty claims the device has to be sent along with the original invoice to:

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boso accepts no responsibility for losses, such as operational disruptions or data losses, which can occur due to the use of this blood pressure monitor.

No guarantee is made that this blood pressure monitor can establish a connection with all Bluetooth-compatible devices.

## Customer service

### Customer service:

Warranty and repair work must be undertaken by trained and authorised personnel. Do not modify this equipment without authorization of the manufacturer.

Send the monitor, carefully packaged and with sufficient postage, to your authorised dealer or directly to:

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## Disposal

Do not dispose of monitors and batteries in the domestic waste.

At the end of its lifespan, the monitor must be taken to a collection site for obsolete electronic items.

Please note that batteries and rechargeable batteries must be disposed of separately (local authority collection site).

Expected operational lifetime of the equipment: 5 years (using the unit 6 times a day).  
Expected operational lifetime of the cuff: 2 years (using the unit 6 times a day).

## Technical data

---

<b>Measurement principle:</b>	oscillometric
<b>Measurement range:</b>	Systolic: 60 to 279 mmHg Diastolic: 40 to 200 mmHg Pulse: 40 to 180 per minute
<b>Cuff pressure:</b>	0 to 299 mmHg
<b>Memory store:</b>	30 measurements
<b>Display:</b>	LCD
<b>Operating conditions:</b>	environmental temperature +10°C to +40°C Relative humidity 15 to 85%
<b>Storage conditions:</b>	environmental temperature -20°C to +60°C Relative humidity 10 to 95%
<b>Power supply:</b>	DC 6 V (4 x 1.5 V Mignon IEC LR 6 alkaline manganese batteries) Alternative special option: DC 6 V power supply unit, Order No. 410-7-150

## Technical data

---

<b>Typical battery life:</b>	500 measurement cycles (depending on inflation pressure and frequency of use).
<b>Battery check:</b>	symbol display in LCD window
<b>Weight:</b>	250 g without batteries
 <b>Dimensions (WxHxD):</b>	96 mm x 68 mm x 130 mm
<b>Classification:</b>	Protection class II (□)
<b>Applied part:</b>	Type BF (⚡)
<b>Protection against solid objects:</b>	IP20
<b>Clinical test:</b>	Accuracy complies with the requirements of ISO 81060 Part 2

## Technical data

---

**Bluetooth**

Bluetooth Version 4.0

**Maximum deviation of cuff pressure measurement:**  $\pm 3$  mmHg

**Maximum deviation of pulse rate display:**  $\pm 5$  %

**Applicable standards:** DIN EN ISO 80601-2-30 „Medical electrical equipment - Part 2-30: Particular requirements for the basic safety and essential performance of automated type non-invasive sphygmomanometers“ .

# Calibration checks - Testing instructions

## (for trained specialist staff only)

Calibration checks – every 2 years at the latest – must be performed by the following institutions or persons:

- Manufacturer
- Trade measurements authorities
- Persons who fulfil the requirements of the legislation governing the operation of medical devices.

### A) Function testing

Function testing of the monitor can only be carried out on a person or with a suitable simulator.

### B) Testing of pressure circuit integrity and deviation of pressure display

#### **Note:**

- a) When the pressure in measurement mode increases above 299 mmHg, the quick release valve is triggered and the pressure circuit opens. When the pressure in test mode increases above 320 mmHg, the measured value display flashes.
- b) For the calibration checks a special test plug must be used. The test plug is available for a fee from the manufacturer.  
A pump ball must also be introduced into the pressure circuit.

# Calibration checks - Testing instructions

## (for trained specialist staff only)

### Testing

- 1.) Remove batteries.
- 2.) Hold start button pressed down and insert batteries.
- 3.) Release start button.  
The monitor is now ready for testing; the current pressure is displayed in the SYS and DIA fields.
- 4.) Perform test for deviation of pressure display and pressure circuit integrity (observe setting time for cuff – at least 30 seconds) in the usual manner.

5.) Switch off monitor by pressing the START button again.

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The screenshot shows the 'Einstellungen' (Settings) screen. At the top, there is a 'Zurück' (Back) button, the title 'Einstellungen', and the 'boso' logo. The main content is organized into three sections:

- Allgemeine Einstellungen**: Contains two time input fields: 'Beginn Vormittag Auswertung' set to '00:00' and 'Beginn Nachmittag Auswertung' set to '12:00'.
- bosomedic system**: Contains a 'Messwertübernahme' (Measurement Transfer) section with two buttons: 'letzter' (last) and 'alle' (all). The 'alle' button is highlighted with a dark background and is circled in red.
- bosogramm system**: Contains a 'Modus' (Mode) section with two buttons: 'singleuser' and 'multiuser'. The 'multiuser' button is highlighted with a dark background.

At the bottom, there is a dark navigation bar with two buttons: 'Abbrechen' (Cancel) with a back arrow icon and 'Speichern' (Save) with a save icon.

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The screenshot shows a mobile application interface for sending reports. At the top, there is a dark header bar with a back arrow and the text "Zurück" on the left, "Bericht" in the center, and the "boso" logo on the right. Below the header, there are three input fields for email recipients: "An:", "Kopie:", and "Email Text:". Each of the first two fields has a magnifying glass icon on the left. The "Email Text:" field is a large, empty text area. At the bottom, there are two buttons: "Als XML senden" and "Als PDF senden", each with an envelope icon above the text.

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(<http://www.boso.de/download/profilmanagerXDHome/>).

The screenshot shows a mobile application interface for sending reports. At the top, there is a dark header bar with a back arrow and the text "Zurück" on the left, "Bericht" in the center, and the "boso" logo on the right. Below the header, there are two input fields for email addresses, each with a magnifying glass icon and the label "An:" above the first and "Kopie:" above the second. Below these fields is a section labeled "Email Text:" with a large, empty text area for composing the message. At the bottom of the screen, there are two dark buttons with white text and envelope icons: "Als XML senden" on the left and "Als PDF senden" on the right.

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**BOSCH + SOHN GmbH u. Co. KG**